INSTRUCTIONS FOR USE

ENG

SOFT CONTACT LENSES

A guide to trouble-free soft contact lens wear.

Daily Wear Soft Contact Lenses (Up to 30 Day Duration) in sterile buffered saline.

SYMBOL REFERENCE GUIDE

SYMBOL	DEFINITION
	Use by date: Refer to packaging
	Batch code: Refer to packaging
	Consult instructions for use
	Sterilised using steam or dry heat
	Do not use if packaging is damaged
	Do not resterilise
	Providing UV protection
	Recyclable
	Use caution
	Temperature range limit
	Distributor details

SYMBOL	DEFINITION
	Manufacturers details
	Indicates the CE conformity marking and the notified body number
	Prescription only product
	Authorised representative in the European community
	Power
	Diameter
	Base curve
	Single sterile barrier system
	Date and country of Manufacture
	Medical Device

Always wash and dry your hands before handling your lenses. Do not use if the packaging or product is damaged. Contact lens, blister and foil to be placed in your general waste bin. Box can be recycled. Please read all instructions prior to use.

LENS INSERTION

For each eye, ensure the lens is not inside out (See Fig. 1 & Fig. 2). Place on your finger to verify the shape. Using your other hand and second finger, hold up the upper lid and hold down the lower lid. Place the lens on the eye (See Fig. 3). Repeat the process for the other eye.

LENS REMOVAL


Look up and hold down the lower lid. Gently pinch the bottom edge of the lens and lift it out of your eye with the pads of your index finger and thumb (See Fig. 4). Repeat the process for the other eye.


Fig. 1 ✓

Fig. 2 ✗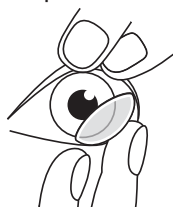

LENS INSERTION
Fig. 3 ✓

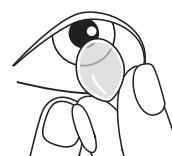LENS REMOVAL
Fig. 4 ✓

DISINFECTION/SOAKING

Clean and disinfect your lenses strictly following the instructions on your solution bottle or as directed by your eye-care practitioner.

If the lenses dry out, follow the disinfection/soaking step above, replace with new lenses or consult your eye-care practitioner.

CONTRAINDICATIONS (Reasons Not To Use)

Do Not Use Contact Lenses when any of the following conditions exist:

- Acute and sub-acute inflammations or infection of the anterior chamber of the eye.
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva or eyelids.
- Severe insufficiency of tears (dry eyes).
- Corneal hypoesthesia (reduced corneal sensitivity).
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient in a solution that is to be used to care for contact lenses.
- Any active corneal infection (bacterial, fungal or viral).
- If eyes become red or irritated.

PRECAUTIONS

Any variation to the following should only be performed under the advice of your eye-care practitioner.

- It is recommended that you replace your lenses frequently and care for them daily. Your eye-care practitioner will recommend a wearing regime. DO NOT change your lens parameters or lens care system without consulting your eye-care practitioner. Follow their recommendations and make regular visits.
- These lenses are intended for wearers ages 8 and up unless advised otherwise by your eye-care practitioner.
- These lenses are designed for daily wear and must be replaced as directed.
- Take care when inserting the lens as a scratched eye is vulnerable to infection. Avoid long nails which harbor bacteria.
- Never use tweezers or other tools to remove lenses from the lens container unless specifically indicated for that use. Pour the lens into the hand.
- Always handle lenses carefully and avoid dropping them. Do not touch the lens with fingernails.
- Discard damaged or scratched lenses immediately.
- Insert lenses before applying make-up. Remove lenses before removing make-up. Use fibre and oil free mascaras. Avoid contact between your lenses or eyes and any cosmetic products including aerosols, lotions, soaps and creams. Water-based cosmetics are less likely to damage lenses than oil-based products.
- If aerosol products such as hairspray are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled. Avoid all harmful or irritating vapors and fumes while wearing lenses.
- Never wear lenses beyond the time recommended by the eye care practitioner.
- Keep lenses out of reach of children.
- If the lens sticks (stops moving) on your eye immediately consult your eye-care practitioner.
- Remove your lenses immediately in the presence of noxious or irritating fumes.
- Consult your eye-care practitioner before using your lenses for any water related activity.
- Only use your lenses for water sports if water-tight goggles are worn.
- Always inform your employer of being a contact lens wearer. Some jobs may require use of eye protection or may require that the you do not wear contact lenses.

WARNINGS

- DO NOT sleep while wearing your lenses.
- DO NOT use household products (e.g. disinfectants) on your lenses.
- DO NOT wear your lenses when using eye medications or eye drops (unless advised to do so by your eye-care practitioner).
- DO NOT touch contact lenses with fingers or hands if the hands are not free of foreign materials, as microscopic scratches of the lenses may occur, causing distorted vision and/or injury to the eye.
- DO NOT use saliva or anything other than the recommended solutions for lubricating or wetting your lenses.
- DO NOT rinse your lenses or lens case with water.
- DO NOT expose your lenses to water while you are wearing them (e.g. showering). Exposing your lenses to water increases the risk of microbial infection.

Problems with contact lenses and lens care products could result in serious injury to the eye. It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of your lenses and lens care products, including the lens case. Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision.

Studies have shown that contact lens wearers who are smokers have a higher incidence of adverse reactions than nonsmokers.

ADVERSE REACTIONS

If you notice any of the following issues with your contact lenses then **IMMEDIATELY REMOVE YOUR LENSES**.

- Eye stinging, inflammation, burning, itching or eye pain
- Decreasing comfort from when you first inserted the lens
- Excessive watering of the eye
- Unusual eye secretions or redness
- Rainbows or halos around objects
- Sensitivity to light
- Dry eyes

If damage to the lens is identified upon inspection, do not put the lens back in your eye. You should discard the lens and insert a new lens into the eye. If the lenses have a foreign body on them and appear undamaged, follow the disinfection/soaking step above, then re-insert them. If the problem continues, remove the lenses and immediately consult your eye-care practitioner.

Excessive wear of contact lenses, particularly overnight wear, is associated with most of the safety concerns and potential adverse reactions with contact lens wear. Problems associated with contact lens wear may affect the eyelid, the conjunctiva, the various layers of the cornea and even the tear film that covers the outer surface of the eye.

For the full set of handling instructions please visit the following www.gelflex.com/handlinginstructions

FOR ALL ENQUIRIES

Manufactured by Gelflex Laboratories.
52 Mulgul Road, Malaga
Western Australia, 6090
www.gelflex.com

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EC REP Gelflex Europe B.V.